

# How to setup a 50 PSI Compressor, tracheostomy, and CPAP/BiPAP



## 50 PSI Compressor Setup

50psi Air compressor is perfect for patients who need humidification. Supplies are listed below in order from compressor to patient as follows:

- ✓ Large volume nebulizer
- ✓ Large bore corrugated tubing
- ✓ Drainage bag
- ✓ Corrugated tubing
- ✓ In-line Neb Tee
- ✓ Oxygen adapter (oxygen tubing is attached at the port and connected to a concentrator)
- ✓ Trach mask

**NOTE:** Large bore corrugated tubing should not exceed 10 ft in length. Cut the tubing at the lowest point and insert the drainage bag to be most effective.



## CPAP/BiPAP Setup

Continuous Positive Airway Pressure (CPAP) and Bi-Level Positive Airway Pressure (Bi-PAP) therapy uses a machine to help a person who has obstructive sleep apnea (OSA) breath more easily during sleep. These machines use increased air pressure to keep the airways open while you sleep. Supplies are listed below in order from machine to patient as follows:

- ✓ 6ft tubing
- ✓ Oxygen adapter (oxygen tubing is attached at the port and connected to a concentrator)
- ✓ Nasal interface (nasal mask, full face mask, nasal pillows, etc)

