



Competency Checklist

XLR8/XLR8 Plus

GENADYNE

Facility Name: _____ Location: _____

Clinician Name and Title: _____ Department: _____

Competency Verified by: _____ Date: _____

XLR8/XLR8 Plus Skill	✓
Attach power cord to pump	<input type="checkbox"/>
Attach and remove canister	<input type="checkbox"/>
Explain canister gel agent and charcoal filter; attach canister to pump	<input type="checkbox"/>
Luer lock system	<input type="checkbox"/>
Turn pump on and off	<input type="checkbox"/>
Screen pump icons	<input type="checkbox"/>
Continuous pressure: 40 mmHg to 230 mmHg	<input type="checkbox"/>
Variable pressure: 0 mmHg to 230 mmHg and 1 minute to 15 minutes	<input type="checkbox"/>
Lock/unlock unit	<input type="checkbox"/>
Alarms: Leak, Blockage, Full Canister, Low Battery, and Critical Low Battery	<input type="checkbox"/>
Demo alarm: settings, troubleshooting, silencing, and alarm log	<input type="checkbox"/>
Dressing kits: foam, thick and thin; gauze; PVA white; silver contact layer; bridge; and spiral	<input type="checkbox"/>
Accessories: Y connector, bed hangar, and carry bag	<input type="checkbox"/>

Competency Checklist

XLR8/XLR8 Plus

Demo Dressings	✓
Choose appropriate foam type <ul style="list-style-type: none"> • Green, white or silver foam for clean wound base 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Green foam for undermining that is not extensive 	<input type="checkbox"/>
<ul style="list-style-type: none"> • White foam for exposed bones, tendons, ligaments, or tunnels 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Silver contact layer for infected wounds 	<input type="checkbox"/>
Prepare surrounding skin <ul style="list-style-type: none"> • Skin barrier wipe to wound edges 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Apply drape to protect surrounding skin 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Consider hydrocolloid to correct/prevent maceration 	<input type="checkbox"/>
Fill wound bed with foam <ul style="list-style-type: none"> • Place contact layer as indicated to protect bones, tendons, and ligaments 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Fill wound base with foam 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Place foam in tunnel with end visible in base 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Place foam in undermined area 	<input type="checkbox"/>
Apply drape and port pad <ul style="list-style-type: none"> • Cover foam and surrounding skin with drape 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Cut quarter-sized hole in drape 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Apply port pad 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Attach port pad tubing to canister tubing 	<input type="checkbox"/>
Bridging dressings <ul style="list-style-type: none"> • When to bridge 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Applying a bridge dressing 	<input type="checkbox"/>
Troubleshooting difficult seals <ul style="list-style-type: none"> • Using stoma paste 	<input type="checkbox"/>
<ul style="list-style-type: none"> • Using paste strips 	<input type="checkbox"/>

For inquiries please call 877-790-5994
or email at order@MasVidaHealth.com.

MVC - Genadyne - XLR8/XLR8 Plus - Competency Checklist - 031423

All trademarks, service marks, and company names are the property of their respective owners. Any products or services offered by MasVida Health Care Solutions, LLC (MasVida) are subject to customer entering into a written customer agreement with MasVida. MasVida is a preferred post-acute distributor of Altra™.